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Reprocessing instruction for holder external canister "Bag"

The holder external canister „Bag“ is an accessory made of stainless steel and is intended for reprocessing. The holder can be attached to the appliances in the same way as described in the current instructions for use.

figure 1: holder external canister "Bag" made of stainless steel with REF 100267-1

Clean and disinfect the surface of the holder external container "Bag" regularly, but at least once a week and when changing patients. Observe the warnings in the instructions for use of the devices.

LIMITATIONS ON PROCESSING	Replace the holder external canister "Bag" in case of visible damage or functional limitations.
PREPARATION BEFORE CLEANING	Remove the holder external canister "Bag" from the device by sliding it upwards along the guide rails.
CLEANING	Remove visible soiling with a damp but not wet, lint-free cloth. Moisten the cloth with clear water (< 40°C / 104 °F). Wipe the entire surface of the holder external canister "Bag" with light pressure. The holder external canister "Bag" is dishwasher-safe and can also be boiled after cleaning.
DESINFECTION	Now disinfect the previously cleaned surface. Spray the surface of the instrument completely. Wipe the entire surface.

The holder external canister "Bag" can be disinfected with disinfectants with active substance "alcohol".

ASSKEA GmbH recommends the following disinfectants:

Surface Disinfectant	Active Substance	Type of Application
Incidin® Liquid (Ecolab)	Alcohol	Ready-to-use solution
Bacillo® AF (Bode Chemie)	Alcohol	Ready-to-use solution

Follow the manufacturer's instructions for the disinfectant.

DRYING	Allow the disinfected surface to dry completely.
MAINTENANCE, INSPECTION AND TESTING	The holder external canister "Bag" should now be free of visible dirt. Attach the holder external canister "Bag" to the device by sliding it from the top into the designated guide.
STORAGE	If the holder external canister "Bag" is not used for a longer period, store it according to the instructions of the device in the section "Storage". There are no special storage conditions for the holder external canister "Bag".

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Die ASSKEA GmbH empfiehlt die folgenden Desinfektionsmittel: